

# Madie®

## *Adjustable Multi-dose Injector*

 **SHL GROUP**  
www.shl-group.com



## *Technical Specifications*

<b>Device Type</b>	Disposable Pen Injector
<b>Primary Container</b>	Up to 3.0mL Cartridge
<b>Dosage Type</b>	Variable   Multi-Dose
<b>Needle Extension</b>	Up to 1/2" or 12.7mm   Subcutaneous
<b>Needle Insertion</b>	Manual
<b>Needle Removal</b>	Manual
<b>Activation Method</b>	Button

Copyright © 2015 SHL Group. All rights reserved. Version 1.1 Information accurate at time of print and is subject to change without notice. Available as standard or customized to client specifications. Contact [info@shl-group.com](mailto:info@shl-group.com) for more information.

# Handling Instructions

**P**



Remove cap, attach needle, turn dial to **04** & press activation button to prime

**1.**



Turn dial to **00** to reset

**2.**



Dial dose. Insert needle into injection site & press activation button

**+**



Recap needle, unscrew & discard in sharps container

## Key Features

- ✓ Advanced reset feature
- ✓ Dial-back dose correction
- ✓ Smooth injection stroke
- ✓ Dosing increments from 10-990µl