



Madie®

3.0 ml

Adjustable multi-dose pen injector

Volume 

Viscosity 

Device Type	Disposable Pen Injector
Primary Container	Up to 3mL Cartridge
Dosage Type	Variable Multi-Dose
Route of Administration	Subcutaneous
Needle Insertion	Manual
Activation Method	Button Activated
Injection Feedback	Audible click at end of injection, viewing window

Handling Instructions



1
Screw on clear cap to attach needle.
Remove the cap



2
Turn dial to the priming dose &
remove yellow cap



3
Press activation button to prime



4
Dial to [00] to reset before dialing
dose



5
Hold against injection site & press
activation button



+

Recap needle & unscrew. Discard in
sharps container

Key Features

- ✓ Advanced reset feature
- ✓ Dial-back dose correction
- ✓ Ergonomic & innovative design
- ✓ Dosing increments from 10-990µL
- ✓ Accommodates high viscosity formulations